

# NO 6 SORBACT® CLINICAL CASE

Post traumatic haematoma / wound to right lower leg.

## WOUND TYPE

Lower leg haematoma

## PRODUCT

Sorbact® Compress

## COUNTRY/LOCATION

DK - Tårnby Municipality

## TREATMENT PERIOD

18 September – 6 November, 2009

## THE PATIENT

A 55 year old female sustained a wound to her right lower limb following a minor trauma, haematoma, wound formation.

## THE TREATMENT

**18 September, 2009** The wound measured 2 x 2 cm in area with a contained purulent exudate. The wound was debrided and cleaned. Sorbact® Compress was used as a wound contact layer with Sorbact® foam supporting to control the exudate. Dressing change occurred 3 times per week. The patient executed the change 2 times per week with nursing staff completing the final change and wound review.

**25 September, 2009** Clinically the wound appeared cleaner with the presence of serous wound exudate only. The management of the wound remained unchanged.

**2 October, 2009** Wound size unchanged, exudate levels continue to decrease and the formation of granulation tissue noted. The management of the wound remained unchanged.

**23 October, 2009** Wound size reduced to 1.5 x 1.5 cm in area, minimal wound secretion and continued formation of healthy granulation tissue.

**30 October, 2009** Patient and clinicians very happy with the progress of wound healing. Management is continued.

**6 November, 2009** Wound size reduced to 0.5 x 0.5 cm in area, frequency of the Sorbact® Compress use changed to twice per week.

