

NO 12 SORBACT[®] CLINICAL CASE

Fungi infection

WOUND TYPE

Mycosis superficialis

PRODUCT

Sorbact[®] Compress
and Sorbact[®] Gel Dressing

COUNTRY/LOCATION

DK - Haderslev Municipality

TREATMENT PERIOD

11 May – 5 August, 2009

WOUND TYPE

Mycosis Superficialis without specification (disease caused by fungi, superficially located).

THE PATIENT

The patient is a 72-year-old highly physically active man. Has during a longer time had circular arterial ulcers which are superficial with a hyperemic border. Has repeatedly been treated with antibiotics

without much effect. The patient has modifications on the nails, reminiscent of fungus. Taken into treatment in nursing clinic and telemedicine in region south is established from the 11th of May, 2009.

THE TREATMENT

11 May, 2009 Visits the nursing clinic for the first time. He is connected to telemedicine, which means that he is monitored closely by the head physicians at the wound clinic in the region. Pictures are taken and text is written. A fungus scraping is made.

Treatment with hydrofiber with silver and foam. Treatment with antibiotics is continued. The wounds are smelling.

19 May, 2009 The yellow necrosis is slowly declining. Debrided at each visit.

22 May, 2009 The wounds are not exuding as much anymore. Product for treatment is changed to silver net.

3 June, 2009 Healing is slow. Attempt to change to Sorbact[®] Compress and Sorbact[®] Gel Dressing to purify since progress is very slow.

22 June, 2009 Healing looks good. All wounds are decreasing. Almost no secretion.

6 July, 2009 Very good effect from Sorbact[®]. Healing is fine.

24 July, 2009 Contact with the dermatologist. Well pleased with the treatment.

5 August, 2009 All wounds are healed. Treatment concluded.

17 August, 2009 The dermatologist writes to e.l. prescribing treatment with tablets Terbinafin and fluid against nail fungus. Must be taken for 3 months.

