

# NO 14 SORBACT® CLINICAL CASE

Burn wound

## WOUND TYPE

Partial thickness burn

## PRODUCT

Sorbact® Surgical Dressing

## COUNTRY/LOCATION

DK - Helsingør Municipality

## TREATMENT PERIOD

14 April – 4 May, 2010

## THE PATIENT

An 83-year-old female.

## THE TREATMENT

**14 April, 2010** The burn to the left thigh measured 2 x 4 cm in size and was assessed as a partial thickness burn. Sorbact surgical dressing 8 x 15cm was applied to the wound and kept in situ for 5 days. The wound displayed signs of infection, including purulent discharge and areas of necrosed tissue.

**19 April, 2010** The necrosed tissue was gone, however a malodorous exudate remained. Limited pain was experienced with dressing change.

**21 April, 2010** Signs of infection diminished. Mild level of exudate with mild pain persisting on each dressing change.

**28 April, 2010** Wound healing continued, minimal exudate noted.

**5 May, 2010** Wound almost completely healed, healthy granulation tissue present.

