

# NO 17 SORBACT® CLINICAL CASE

Trauma wound to the right lower leg

## WOUND TYPE

Trauma wound

## PRODUCT

Sorbact® Compress

## COUNTRY/LOCATION

DK - Sorø Municipality

## TREATMENT PERIOD

18 February – 14 April, 2010

## THE PATIENT

An elderly woman.

## THE TREATMENT

**11 February, 2010** Previous treatment led to maceration of the peri-wound tissue. Treatment commenced using Sorbact® Gel and a cotton based secondary dressing. Dressing change frequency was every second day given low levels of exudate. The wound contained non-viable, difficult to remove tissue. No pain was associated with each dressing change.

**17 March, 2010** The wound was healing well. Non-viable tissue removed via autolytic debridement. Wound management changed to the Sorbact® Compress dressing with a change frequency of 2 times per week.

**23 April, 2010** Complete wound healing reported. No antibiotics used in support of the wound care in this case.

