

NO 13 SORBACT[®] CLINICAL CASE

Complex trauma wounds

WOUND TYPE

Trauma wound

PRODUCT

Sorbact[®] Compress

CONTRY/LOCATION

DK - Sorø Municipality

TREATMENT PERIOD

19 February – 23 April, 2010

THE PATIENT

An elderly female.

THE TREATMENT

19 February, 2010 Prior to commencing management of the wounds with Sorbact[®], the patient was experiencing considerable pain upon dressing change. The wounds were heavily exuding and requiring a second daily dressing change. Sorbact[®] Compress was used as a primary contact layer for both wounds with the secondary dressing selected as a super absorbant with overlying compression applied.

20 March, 2010 At this point in management, the level of exudate was reduced, however, mild erythema of the surrounding tissue persisted. The wounds appeared to be healing well with significantly less pain associated with each dressing change. The Sorbact[®] Compress was very simple to remove. The wound management remained unchanged.

9 April, 2010 The frequency of dressing change was extended to twice a week with good wound healing demonstrated.

16 April, 2010 The wound to the left is leg completely healed. Management of the right leg wound continued as the wound was almost healed.

23 April, 2010 Both wounds are now completely healed.

